



**STRAUSS & CO.**

INDUSTRIAL DIAMONDS LTD

Page 1 of 2

## **"Strauss & Co." General instructions for use-Magic Strip**

### **General instructions**

- For the safety of user and patient, it is highly recommended that only qualified dentists should work with the Magic Strip.
- Strauss & Co.'s magic strips designed for several uses and are autoclavable.
- The strips are designed to work manually in the interproximal gap.
- Avoid touching soft tissue with the strips.
- Sterilize the magic strip before use.

### **Sterilization instruction**

#### **Instructions for cleaning and sterilizing the magic strip**

Disinfect and sterilize the strips before every use.

#### **Cleaning:**

- Clean the strips by dipping them in disinfection solution that contains a corrosion inhibitor for a minimum of 10 minutes.
- If needed - use brush to clean thoroughly.
- Rinse several times to remove all the disinfection solution.

#### **Cleaning with Ultrasound:**

- Strips may be ultrasonically cleaned.
- A cycle of 5 minutes is recommended, using a general purpose cleaner.
- Rinse several times.

#### **Sterilizing magic strip – autoclave:**

- Place the strip in sterilization bags.
- Autoclave the strip for a minimum of 10 minutes at a temperature of 135°C.

Following sterilizing or cleaning methods, dry the strips and store in a clean and moisture free environment.

#### **Note:**

- Use sterilizing and cleaning devices and materials according to the manufacturers recommended procedure.
- It is the responsibility of the user to ensure that sterilization is effective.



**STRAUSS & CO.**  
INDUSTRIAL DIAMONDS LTD

Page 2 of 2



MedNet GmbH  
Borkstrasse 10  
48163 Muenster  
Germany

Ed 0818